



Hammerlit Shuttle-System



Shuttle-System: **113**
function

Transport Carts **114**

Shuttle Racks **114-115**

Dollies and Pedestals **116**



Docking of Shuttle Cart

With a 2-point linking mechanism the shuttle cart is easily and safely connected to the Shuttle transport cart.



Approach the cart with an handle ...



... until it is stopped ...



... and shift into the cabinet. It snaps in automatically.

Unlinking the Shuttle Cart

For unlinking simply use the locking bar at the handle and swivel the shuttle cart away.

Pass over the Shuttle rack

After docking, the shuttle can be shifted in (without changing the position):



... and shift ...



... until it snaps in.



Simply lift the locking bar ...

Take over the Shuttle

To get the shuttle rack out of the cart, the procedure has to be done vice versa.

Simply lift the locking bar again and pull the shuttle out until it snaps in on the shuttle cart automatically.



RGEE2x3SH

Elements to run with Shuttle Rack

- silent and stable elements to guide 1 or 2 Shuttle Racks
- Dolly and Shuttle snap in at the frame of transport cart automatically



RGEE2x3SH, 2 x EGSTE3E60



Elements for Shuttle Rack

Capacity (STE)	Dimensions WxDxH (mm)	# Model	matching Shuttle Rack	matching Dollies/ Pedestrals
3 x 2	980 x 725 x 1365	RGEE2x3SH	EGSTE3E60, EGSTE3	UEWSTE3
6 x 1	980 x 725 x 1365	RGEE6SH	EGSTE6E60, EGSTE6	UEWSTE6
9 x 1	1255 x 725 x 1365	RGEE9SH	EGSTE9E60, EGSTE9	UEWSTE9
2 x 6 Trays*	1255 x 725 x 1365	RGEE2xISO1SH	EGISO1E60, EGISO1	UEWISO1
12 Trays*	1125 x 725 x 1365	RGEEISO2SH	EGISO2E60, EGISO2	UEWISO2

*600 x 400 x 100 mm

Shuttle System: Shuttle Rack

Sterile Goods



EGSTE9E60 with 9 pair of slides CES60

Shuttle Racks

- Material 1.4301/ AISI 304 Stainless steel
- 25 x 25 mm square-profile tubing - construction, seamless welded
- Unlocking-bar dimensions in full width of rack, which allows front and side access for handling
- 4 conductive castors, Ø 75 mm
- autoclavable



Capacity	Dimensions W x D x H (mm)	Model #
3 x 1 STE	355 x 600 x 1050	EGSTE3E60
6 x 1 STE	685 x 600 x 1050	EGSTE6E60
9 x 1 STE	1015 x 600 x 1050	EGSTE9E60

Shuttle System: Shuttle Rack



EGSTE9 with 2 shelves CEGSTE9

Shutte Rack with wire shelves

- 2 adjustable and one fixed wire shelf
- option: with dividers **CEGSTEFT**



Capacity	Dimensions W x D x H (mm)	Model #
6 x 1 STE or 3x1 ISO-Tray	650 x 600 x 1050	EGSTE6
9 x 1STE or 6x1 ISO-Tray	950 x 600 x 1050	EGSTE9



EGISO1 with 6 pair of slides CESU60

Shuttle Rack with U-Slides, height adjustable

- with 6 resp. 12 pair of U-Slides
- optional with arrester (stop) bar
- optional with security bar



Capacity	Dimensions W x D x H (mm)	Model #
1 x 6 ISO-Trays 40/60	455 x 600 x 1050	EGISO1
12 ISO-Trays 40/60	885 x 600 x 1060	EGISO2

Sterile Goods



UEWSTE9SF



RGEE2x3SH, 2 x EGSTE3E60, UEWSTE3



UEWISO1

Dollies

- Stainless Steel AISI 304
- With guidance and anchoring mechanism to accommodate one or two Shuttle Racks
- The dolly is fitted with a two-point docking mechanism compatible with corresponding Transport Cart RGEE SH or pedestal PF STE
- Dolly can be easily uncoupled from Transport Cart using the ergonomically designed release mechanism built into the push-bar
- Solid plastic bumpers on three sides
- Four 125 mm conductive swivel castors, two with double stoppers
- Suitable for machine washing and autoclave in the „SF“ version

Matching Shuttle Rack	Dimensions W x D x H (mm)	Model
EGSTE3, EGSTE3E60	445 x 640 x 945	UEWSTE3
EGSTE9, EGSTE3E90	1110 x 640 x 945	UEWSTE9

Matching Shuttle Rack	Dimensions W x D x H (mm)	Model
EGSTE6, EGSTE6E60	785 x 640 x 945	UEWSTE6
EGISO1, EGSTE3	555 x 640 x 945	UEWISO1

Shuttle System: Pedestals



PFSTEIS02

Pedestals

- Stainless Steel AISI 304
- With a guidance and anchoring mechanism to accommodate one or two transport modules
- With docking mechanism
- Four conductive height-adjustable feet
- Suitable for machine washing

Quantity Shuttle Rack	Dimensions W x D x H (mm)	Model
1	530 x 685 x 300	PFSTE3
2	895 x 685 x 300	PFSTE2x3
1	840 x 685 x 300	PFSTE6
1	1170 x 685 x 300	PFSTE9

Quantity Shuttle Rack	Dimensions W x D x H (mm)	Model
1	630 x 685 x 300	PFISO1
2	1180 x 685 x 300	PF2xISO1
1	1060 x 685 x 300	PFISO2